


PREG-TONE® SERIES 6 ANIMAL SELECTION CHART

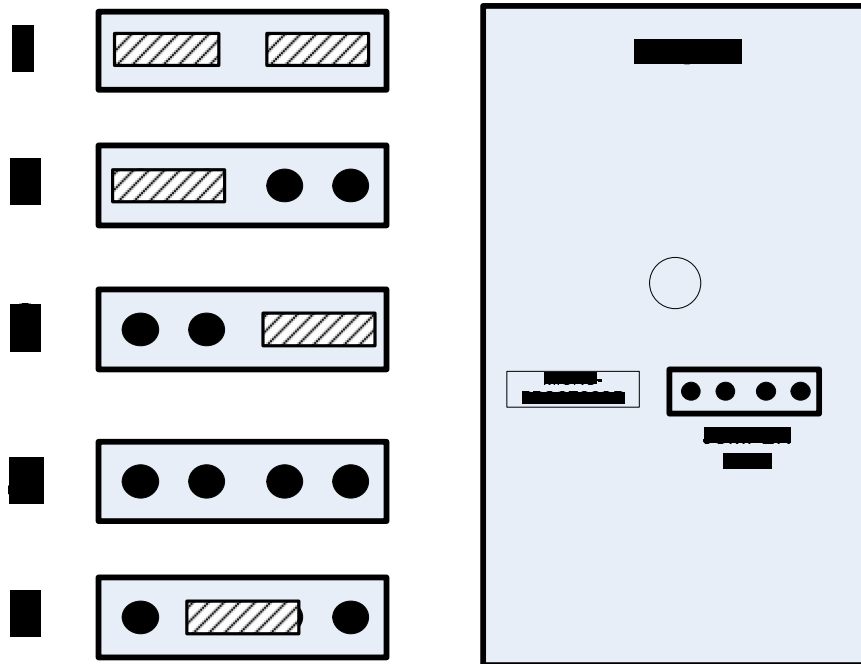

JUMPER SETTINGS

Directions:

Hold instrument with belt clip side down. Remove screw and cover. Install or remove jumpers so that for a desired animal, the figure indicated for the animal, matches diagram number above. Unused jumpers may be installed on any unused single pin. These are suggested settings. Animals vary and you may find that another setting works better for your animals.

<u>Animal</u>	<u>Figure</u>
PIG	1
LLAMA	1
SHEEP	2
MEAT/DAIRY GOATS	2
ALPACA	2
LARGE RABBIT/FOX	3
PYGMY GOAT	4
COW (>90-DAYS)	1
MARE (>90-DAYS)	1
MINIATURE MARE	1
DONKEY	1

<u>Animal</u>	<u>Figure</u>
MINIATURE DONKEYS	4
CUSTOM OR SPECIAL	5

Technical Note:

Windows open at following times: 1=180us,
2=130us, 3=85us, 4=115us