

CORRECTION: The SONO-GRADER GUIDES prior to v.4.7a contain incorrect information in regard to the section on EDITING . The following is the correct information.

EDITING

The “**C**” (correction key) can be used to delete the number just entered. When the cursor is at the **AN#** position, pressing the **left** arrow key will present an option to redo the previous record. (“**Do the PREVIOUS animal again?**”.)

If selected, data for the previous animal, will be cleared and **IDENTIFICATION** screen will then display the Record number for previous animal. If not selected, i.e. any key other than “**1**” is pressed, the **IDENTIFICATION** screen is re-displayed.

For example: You are currently working with Record **#50**, and have entered data for **AN#** and **PEN#**. The cursor is now in **CODE#** field. You discover that you made an error in regard to the animal in Record **#49**. You must complete **ALL** of the entries for Record **#50**. The cursor is now in the **AN#** field for Record **#51**. Press the **left** Arrow key to delete and redo Record **#50**, and once again to delete and redo Record **#49**.

Pressing the “**C**” key after storing **P1**, **P2**, or, **P3**, takes you back to the beginning of the respective screens. But, since the **P2** reading and **Loin** reading are related, pressing the “**C**” key after doing the **Loin** measurement, takes you back to the beginning of the **P2** screen.